

# DYMACARE®

everyday care

## PATIENT CLEANSING

Eliminate Tap Water & Basins  
and Standardize Care.



## BASINS ARE A PROVEN RISK OF INFECTION


- Published studies have shown that there is risk of cross-contamination while bathing patients using a wash basin and cloth.
- Hospital acquired infections (HAI) have been linked to the use of wash basins
- Hospital tap water has the potential to be contaminated.
- Wash basins are used several times a day, often with different patients, increasing the risk of cross-contamination.



**1.** Most common contamination is likely patients' own bacteria.



**2.** When the basin is emptied micro-organisms are retained within the basin.

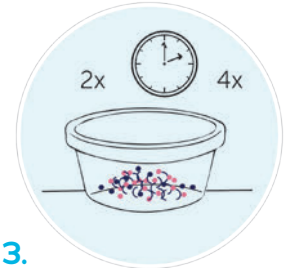


**3.** Micro-organisms thrive in wet and warm conditions, multiplying over time.



**4.** When the basin is refilled, micro-organisms are viable within the water.



**5.** Using the water from the basin the micro-organisms are picked up.

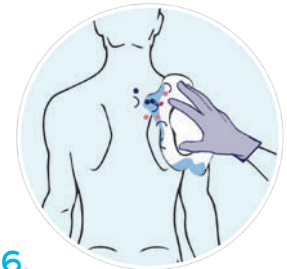

**6.** The next patient is cleaned with the contaminated water.

## DYMACARE<sup>®</sup> PATIENT BATHING REMOVES THE NEED FOR BASINS



Reduces HAI risk factors



Better health outcomes



Improves patient experience



Improves friends & family scores

## OUR DYMACARE<sup>®</sup> PATIENT CLEANSING RANGE:



### WASH MITTS

- Easy to use
- Allows nurses to have one hand free



### BED BATH WIPES

- Easy to use
- Eliminates need for water & rinsing
- Makes bathing quicker



### SHAMPOO CAP

- Easy to use
- Eliminates need for water
- No need to move patient from room



## CONSISTENCY & QUALITY

- Consistent care from cloth to cloth, patient to patient
- Easy to follow, standardised protocol
- Predictable per bath costs



## BETTER SKIN CARE

- Cleans, moisturises & conditions the skin
- pH balanced, dermatologically tested
- Thick & soft cloths
- No harsh ingredients



## COST EFFECTIVE

- No laundering of towels nor cloths required
- Faster preparation & quicker bathing saves staff time
- The reduction in cross-contamination events reduces HAI, saving time and extra costs



## COMFORT

- Reduced need of turning & lifting patients
- Less loss of body heat
- Soft non-woven cloths gentle to vulnerable patients skin
- Improves patient cleanliness

# DYMACARE® ALL-IN-ONE BATHING SYSTEM



## MORE HYGIENIC

- Remove dirty basins and concerns about tap water quality
- Individual patient products, no contamination between person-to-person
- Every cloth is easy to take out and free from micro-organisms

## PRODUCT BENEFITS:



No need to rinse



Can be used for full body cleansing



Dermatologically tested



Thick & strong wipes - gentle to the touch but tough enough for all cleaning needs



Z folded - clean, safe, simple



Non-GM and have not been tested on animals



Do not contain parabens or alcohol



Can be warmed up in a microwave

# 89%



Removal of bath basins has been shown to reduce CAUTI (catheter-associated urinary tract infections) infection rates by 89%.<sup>3</sup>



### References:

1. D. Marchaim et al. Hospital bath basins are frequently contaminated with multidrug-resistant human pathogens, American Journal of Infection Control 2012;40:562-4
2. L. Ngodeskiou, Lars E. Henningsen, Britta Hordam - Elderly patients' and nurses' assessment of traditional bed bath compared to prepacked single units - randomised controlled trial <https://www.researchgate.net/figure/Nurses-bath-type-preferences.tb3.265389491>
3. Stone S, et al. Removal of bath basins to reduce catheter-associated urinary tract infections. Poster presented at APIC 2010, New Orleans, LA, July 2010



## BED BATH WIPES | UNFRAGRANCED

wipe size 30 x 20 cm

Unscented skin cleansing wipes enriched with Aloe Vera. Gentle, dermatologically tested, pH balanced formula, suitable for all skin types, including sensitive skin. Excellent for bed bath.

8 wipes/pack | 30 packs/case

REF | DY200



## BED BATH WIPES | UNFRAGRANCED

wipe size 30 x 20 cm

Unscented skin cleansing wipes enriched with Aloe Vera. Gentle, dermatologically tested, pH balanced formula, suitable for all skin types, including sensitive skin. Excellent for bed bath.

5 wipes/pack | 48 packs/case

REF | MA910



## BED BATH WIPES | FRAGRANCED

wipe size 30 x 20 cm

Lightly scented skin cleansing wipes enriched with Aloe Vera. Gentle, dermatologically tested, pH balanced formula, suitable for all skin types. Excellent for bed bath.

8 wipes/pack | 30 packs/case

REF | DY200F



## WASH MITTS | UNFRAGRANCED

wipe size 22 x 16 cm

Unscented skin cleansing mitts enriched with Aloe Vera. Gentle, dermatologically tested, pH balanced formula, suitable for all skin types, including sensitive skin. Excellent for bed bath.

8 mitts/pack | 30 packs/case

REF | DY400



## SHAMPOO CAP | FRAGRANCED

one Size

Cleanses and cares for hair and scalp. With moist mild shampoo and conditioner. Gentle, dermatologically tested, pH balanced formula, suitable for all hair types. No need to rinse, simply towel dry.

1 Cap | 30 packs/case


REF | DY300



## DIRECTIONS FOR USE:

One pack is designed for use by one patient and will enable them to cleanse their entire body. Nursing staff should assist if the patient is unable to use the wipes themselves. The wash cloths can be warmed or used straight from the pack. After the cleanse, allow your skin to air dry, no need to rinse off.

### Wipes & mitts:

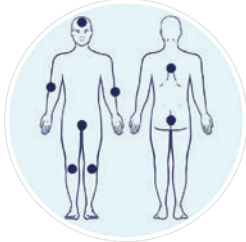


**1.** For the 8 cloth packets, conduct full body cleansing as indicated in the diagram above.



**2.** Clean the face, neck and hands.




**3.** Use one wipe for the chest area. Use another wipe to clean the back.

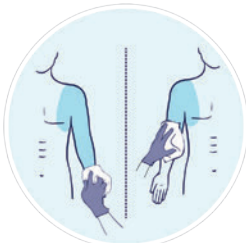

**4.** Use one wipe to clean the arm, hand and the armpit area. Repeat same for the other arm using a new wipe.



**5.** Use one wipe to clean each leg.

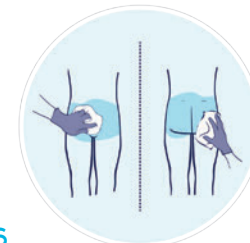

**6.** Use one wipe to clean groin, perineum and buttocks.

### Shampoo caps:



**1.** Put on gloves prior using the product. There is one single use cap in a pack. Open the pack and place it over the patient's head.



**2.** Massage it well and leave it for a few minutes for the product to cleanse and condition the hair.



**3.** Remove it. No need to rinse. Discard used cap and gloves to bin.

## INGREDIENTS

**Fragrance free wipes & mitts:** Aqua, Propylene Glycol, Aloe Barbadensis (Aloe Vera) Leaf Gel, Phenoxyethanol, Polysorbate 80, Sodium Benzoate, Capryl/ Capramidopropyl Betaine, Bis-PEG/PPG-20/20 Dimethicone, Disodium EDTA, Citric Acid.

**Fragranced wipes:** Aqua, Propylene Glycol, Aloe Barbadensis (Aloe Vera) Leaf Gel, Phenoxyethanol, Polysorbate 80, Sodium Benzoate, Capryl/ Capramidopropyl Betaine, Bis-PEG/PPG-20/20 Dimethicone, Disodium EDTA, Parfum (hexyl cinnamal, citronellol), Citric Acid.

**Shampoo caps:** Aqua, PEG-40 Glyceryl Cocoate, Phenoxyethanol, Sodium Coceth Sulfate, Sodium Benzoate, Amodimethicone, Tetrasodium Glutamate Diacetate, Parfum, Trideceth-12, Cetrimonium Chloride, Citric Acid.



PEARMINE  
Health



+44 (0) 1280 416 070



info@dymacare.com  
www.dymacare.com



Pearmine Health Ltd,  
Unit 5, The Sidings,  
Top Station Road  
Brackley, NN13 7UG, UK.

DYMACARE® is a registered trademark of Pearmine Health Ltd.  
RP-Advena Ltd, Tower Business Centre, 2nd Flr, Tower Street, Swatar, BKR 4013, Malta.

Made in the UK